


Now that you're ready to start treating patients, here is a step-by-process for you to follow.

Step 1: Ensure Your 3M™ Oral Care Portal Account is Active

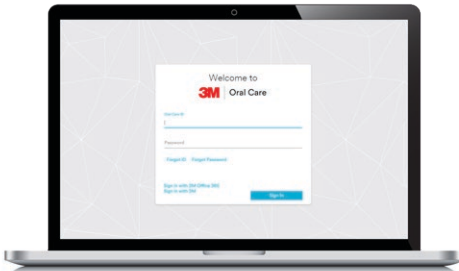


Being prepared will help your first 3M™ Filtek™ Matrix procedures go smoothly.

- Accept the 3M™ Oral Care Portal Invitation
- Familiarize yourself with the portal and set up any staff
- Be sure to pair scanners if available or request shipping materials for physical impressions prior to the records appointment
- Review required scans and photos


Step 2: Collect Records at the Appointment



Photos and scans will help improve your ability to deliver the desired results.

- Take full arch scans or impressions including upper arch, lower arch, and bite registration
- Take required photos 

Step 3: Complete Digital Prescription on the 3M™ Oral Care Portal

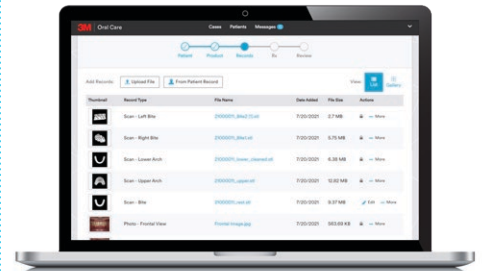

- Upload patient records in the 3M™ Oral Care Portal
- Enter patient information and complete the prescription

Step 4: Review Treatment Design and Approve Restoration



Production of your Filtek Matrix will not begin before the design is approved.

- Assess the Filtek Matrix digital mock-up
- Approve or make changes

Step 5: 3M Produces Filtek Matrix



- The 3D- printed matrix will be ready to use when it arrives
- Production typically takes less than two weeks
- In the meantime, review training resources

Step 6: Transform Your Patient's Smile



- Schedule and complete the procedure
- Be sure to check Oral Care Portal for case updates
- Review case notes and tips & tricks documents for technique guidance

Need help?

For case specific questions, message 3M directly in the Oral Care Portal or call the 3M Oral Care Customer Care team for troubleshooting and technical support: **1-800-634-2249**