




Quick start guide

Adding bite blocks and bite ramps to Clarity™ Digital Bonding

01

Using your credentials, log into Clarity™ Portal.

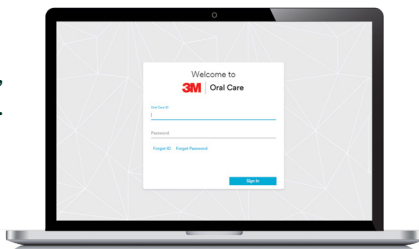

02

Select “**New Order**” in the upper right corner.



03

Choose “**New Patient**” and fill in “**New Patient Information**”. Select “**Next**”.

04

Select “**Digital Bonding Tray**” from the product drop down menu.

05

Review case records and select “**Next**” to continue to the prescription form.

06

Fill in Prescription form.

07

Doctors can request bite ramps and/or bite blocks when one of the following **“Treatment Goals to Address”** boxes is selected for an order with a Digital Bonding Tray arch(es):

- Class III
 - Bite ramps may be placed on the lower arch only if the lower arch is Digital Bonding
 - Bite blocks offered for both arches
- Anterior open bite
 - Bite blocks offered for both arches
- Deep bite
 - Bite ramps offered for the upper arch only
- Posterior crossbite
 - Bite ramps offered for upper arch only
 - Bite blocks offered for both arches
- Brodie bite
 - Bite ramps offered for upper arch only
 - Bite blocks offered for both arches
- Anterior crossbite
 - Bite ramps offered for upper arch only
 - Bite blocks offered for both arches

If you wish to place bite ramps, select **“Yes”**. Select the teeth on which you would like bite ramps placed.

If you wish to place bite blocks, select **“Yes”**. Select the teeth on which you would like bite blocks placed.

Select **“Next”**.

Bite ramps? ⓘ

☐ No
 ☒ Yes

Place on these teeth:

LR

4.3

4.2

4.1

3.1

3.2

3.3

LL

Bite blocks? ⓘ

☐ No
 ☒ Yes

Place on these teeth:

UR

1.7

1.6

1.5

1.4

2.4

2.5

2.6

2.7

UL

LR

4.7

4.6

4.5

4.4

3.4

3.5

3.6

3.7

LL

Place on:

☐ Buccal cusp
 ☒ Center of the occlusal surface
 ☐ Lingual cusp

08

Once the Rx is submitted, you will see the selected bite block and bite ramp options on both the review page and the Rx summary PDF.

Treatment Goals to Address


Addressed in Initial Tx: Anterior crossbite
 Bite ramps: No
 Bite blocks: Yes
 Place on these teeth: UR7; UR6; UL6; UL7
 Place on: Center of the occlusal surface

Treatment Goals to Address

Addressed in Initial Tx: Anterior crossbite
 Bite ramps: Yes
 Place on these teeth: UR1; UL1
 Bite blocks: No

09

Once returned, your Tx Design will be viewable in the Tx Designer. You may view or suppress the bite blocks and bite ramps, and they may also be removed in revision mode.

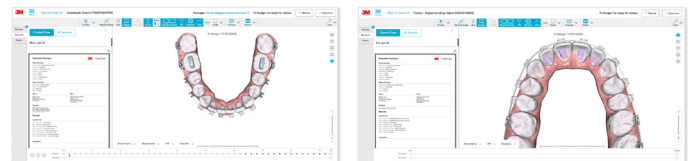

10

Finally, the Tx Design Summary PDF will be updated to show the presence of bite blocks and/or bite ramps with new icons and the Digital Bonding Tray details.

3M Oral Care
 Tx Design Summary
 Patient: Charles Contreras | DOB: 1980-01-10 | Case ID: 5152111 | Order ID: V101002
 Doctor: Brad Knoll | Approved: 2025-01-14 | Handwritten: Patient

Upper: SmartClip™ Aligners (Flex 1-3), Force 32-47
 Lower: SmartClip™ Advanced, Victory Series™ Smart Brackets

IPR Treatment Plan (No IPR Prescribed)

3M Oral Care
 Tx Design Summary
 Patient: Charles Contreras | DOB: 1980-01-10 | Case ID: 5152111 | Order ID: V101002
 Doctor: Brad Knoll | Approved: 2025-01-14 | Handwritten: Patient

Lower Arch Digital Bonding Tray Detail

Tooth	Block	Bracket/Rescued Tubes	Hook	Slot Size	Part #
LR6		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR7		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR8		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR9		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR10		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR11		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR12		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR13		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR14		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR15		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR16		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR17		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR18		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR19		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR20		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR21		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR22		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR23		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR24		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR25		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR26		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR27		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR28		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR29		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR30		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR31		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR32		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR33		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR34		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR35		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR36		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR37		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR38		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR39		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR40		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR41		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR42		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR43		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR44		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR45		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR46		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR47		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR48		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR49		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR50		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR51		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR52		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR53		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR54		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR55		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR56		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR57		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR58		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR59		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR60		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR61		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR62		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR63		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR64		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR65		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR66		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR67		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR68		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR69		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR70		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR71		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR72		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR73		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR74		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR75		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR76		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR77		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR78		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR79		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR80		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR81		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR82		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR83		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR84		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR85		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR86		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR87		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR88		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR89		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR90		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR91		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR92		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR93		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR94		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR95		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR96		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR97		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR98		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR99		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014
LR100		SmartClip™ Advanced™ Rescued™ 19 Single Non-Connectable		250	A100-014



Solventum Dental Solutions
1-800-634-2249
solventum.com

For additional support, please call 1-800-423-4588

© Solventum 2025. Solventum, the S logo, Clarity and SmartClip are trademarks of Solventum or its affiliates. 3M and the 3M logo are trademarks of 3M. All other trademarks are the property of their respective owners.