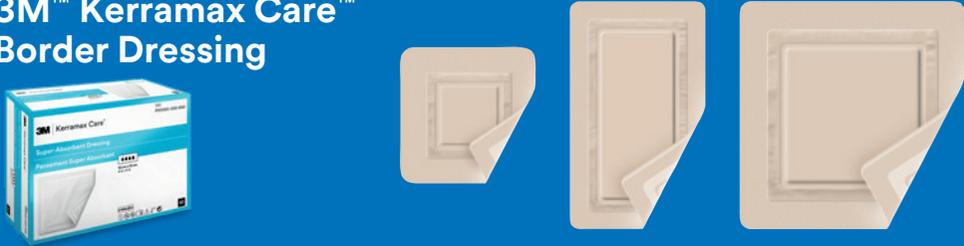


When to select a foam vs superabsorbent dressing

1 Select the level of exudate expected until next scheduled dressing change
Assess if the skin is fragile or non-fragile

	Low to high exudate? 	Moderate to very high exudate? 
Fragile skin	<p>3M™ Tegaderm™ Foam portfolio</p>  <p>3M™ Tegaderm™ Silicone Foam Border Dressing 3M™ Tegaderm™ Silicone Foam Dressing 3M™ Tegaderm™ High Performance Foam Non-Adhesive Dressing</p> <p>Dressing itself indicates the right time for change.</p>  <p>Not ready Not ready Ready to change</p>	<p>3M™ Kerramax Care™ Super-Absorbent Dressing</p>  <p>10cm x 10cm 13.5cm x 13.5cm 10cm x 22cm 20cm x 22cm 20cm x 30cm 20cm x 50cm</p>  <p>92ml* 177ml* 221ml* 404ml* 723ml* 1,310ml*</p>
Non-fragile skin	<p>3M™ Tegaderm™ High Performance Adhesive Foam Dressing</p> 	<p>3M™ Kerramax Care™ Border Dressing</p> 

2 Protect Peri-wound area with 3M™ Cavilon™ No Sting Barrier Film

3 Consider compression therapy when applicable (recommend non-bordered dressing options)

Ordering information

Cat. No.	Product	Size	Items/box
3M™ Tegaderm™ Silicone Foam Dressing			
90631	Non-bordered Dressing	10 × 10.8cm	10
90632	Non-bordered Dressing	15 × 15cm	10
90640	Bordered Dressing	7.5 × 7.5cm	10
90641	Bordered Dressing	10 × 10cm	10
90642	Bordered Dressing	15 × 15cm	10
90643	Bordered Dressing	5 × 5cm	10
90646	Heel and Contour	16.5 × 16.5cm	5
90647	Small Sacral	15 × 17cm	10
90648	Large Sacral	18.5 × 22cm	5

Cat. No.	Foam pad size	Size	Items/box
3M™ Tegaderm™ High Performance Foam Adhesive Dressing			
90610 (square)	5 × 5cm	8.8 × 8.8cm	10
90611 (oval)	6 × 7.6cm	10 × 11cm	10
90612 (square)	10 × 10cm	14.3 × 14.3cm	10
90613 (oval)	10 × 11cm	14.3 × 15.6cm	5
90614 (mini oval)	3.1 × 3.8cm	6.9 × 7.6cm	10
90615 (square)	2.5 × 2.5cm	6.9 × 6.9cm	10
90616 (oval)	14 × 17.1cm	19 × 22.2cm	5
90619 (heel/elbow)	7.6 × 7.6cm	13.9 × 13.9cm	5

Cat. No.	Size	Items/box
3M™ Tegaderm™ High Performance Foam Non-Adhesive Dressing		
90600	5 × 5cm	10
90601	10 × 10cm	10
90602	10 × 20cm	5
90603	20 × 20cm	5
90604	8.8 × 8.8cm	10

Item code	Size	Items/box
3M™ Kerramax™ Care Super-Absorbent Dressing		
PRD500-025	5 × 5cm	10
PRD500-050	10 × 10cm	10
PRD500-050-B50	10 × 10cm	50 (bulk pack)
PRD500-120	10 × 22cm	10
PRD500-120-B50	10 × 22cm	50 (bulk pack)
PRD500-240	20 × 22cm	10
PRD500-240-B30	20 × 22cm	30 (bulk pack)
PRD500-380-B10	20 × 30cm	10
PRD500-380-B30	20 × 30cm	30 (bulk pack)
PRD500-600-B10	20 × 50cm	10
3M™ Kerramax™ Care Border Dressing		
CWL1000	16 × 16cm	5
CWL1001	16 × 26cm	5
CWL1002	26 × 26cm	5

*Lodhi, N. & Warde, D. Free-swell absorbency and absorbency under compression testing of Kerramax Care (all SKUs) – internal report CHC R839. Crawford Healthcare Ltd. Knutsford, UK: 2019.

3M, Kerramax Care and Tegaderm are trademarks of 3M Company. © 3M 2022. All rights reserved. OMG288094.