

VH2O2 indicator	Unexposed	Acceptable exposed color range		
3M™ Comply™ Indicator Tape 1228				

After processing, indicator stripes turn to pink.

Unprocessed	Acceptable processed PASS	Unacceptable processed FAIL
3M™ Attest™ Tri-Metric Chemical Indicator 1348/1348E		

If a color change to pink has entered the ACCEPT zone it is considered a pass.

If a color change to pink has not entered the ACCEPT zone, it is considered a fail.

STEAM indicator	Unexposed	Acceptable exposed	
3M™ Comply™ Indicator Tape 1322			
3M™ Comply™ Indicator Tape 1355			

After processing, indicator stripes turn dark-brown/black.

Unprocessed	Acceptable processed PASS	Unacceptable processed FAIL
3M™ Attest™ Steam Chemical Integrator 1243		

After processing, indicator bar crosses into the ACCEPT region.

If the color bar is in the REJECT region or on the line, it is considered a fail.

Solventum Medical Surgical

© Solventum 2025. Solventum, the S logo, Attest and other trademarks are trademarks of Solventum or its affiliates. 3M, the 3M logo and Attest are trademarks of 3M. Other trademarks are the property of their respective owners.