



3MTM AttestTM Vapourized Hydrogen Peroxide Tri-Metric Chemical Indicator, Type 4

Unprocessed



3MTM AttestTM Vapourized Hydrogen Peroxide Tri-Metric Chemical Indicators, Type 4 comply with ISO 11140-1:2014.


Available in Canada from

3M Canada Company
Medical Solutions Division
300 Tartan Drive
London, Ontario N5V 4M9
Canada
1-800-364-3577
3M.ca


3M Company
2510 Conway Avenue
St. Paul, MN 55144
USA
1-800-228-3957

© 2022 3M. All rights reserved. 3M and the other marks shown are marks and/or registered marks. Unauthorized use prohibited. 3M marks used under license in Canada. 2204-23776. CA_EN_70-2011-8057-0 (02/22)


Processed

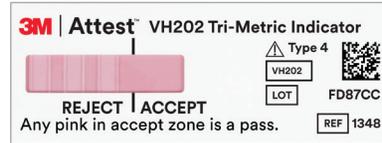


Accept

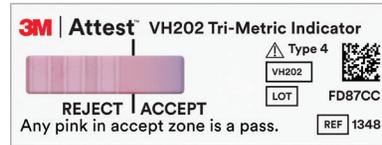

Accept

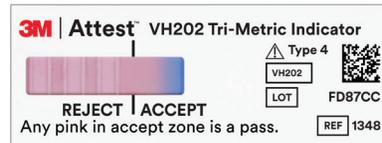

Accept

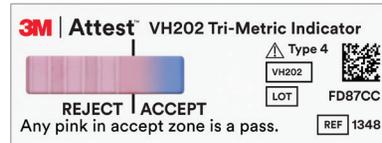

Accept



If a colour change to pink has entered the Accept zone it is considered a pass.

Processed



Reject



Reject



Reject



If a colour change to pink has not entered the Accept zone, it is considered a fail.

Please reference the instructions for use for additional information.