

3M™ Coban™ 2 Lite Two-Layer Compression System


Scrotum and penis application

Materials and positioning

- ▶ Select the 7.5cm or 10cm lite comfort foam layer and lite compression layer for the scrotal application depending on the size of the patient

Application of comfort foam layer for scrotum: layer 1

- ▶ Apply this layer with the foam side against the skin with slight stretch to conform
- ▶ Cover the skin with as thin a layer as possible with no gaps

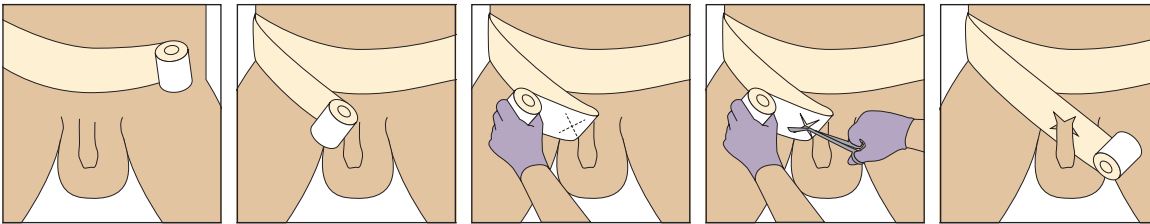


Step 1: Begin the application with a circular winding around the hips centering over the mons pubis.

Step 2: Bring the winding distally toward the base of the penis.


Step 3: To make the penile opening, fold the comfort foam layer back at the base of the penis. Make two cuts in an X pattern at the pinched fold. **Note: In case of penectomy or short penis, the fold and cut slits should be made at the area of the external meatus.**

Step 4: Guide the penis through the hole in the comfort foam layer. If required, extend the cut slits for comfort.

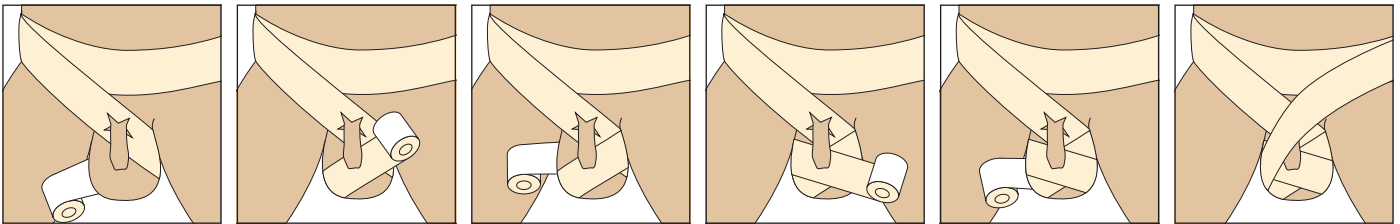

Step 5: Proceed with bandage distally bringing the winding under the scrotum. Come across the top in an upward direction. Continue windings to cover all exposed areas in as thin a layer as possible.

Step 6: Once scrotum is covered, bring the bandage laterally towards the opposite hip. At the lateral side, press, mould, and cut the bandage.

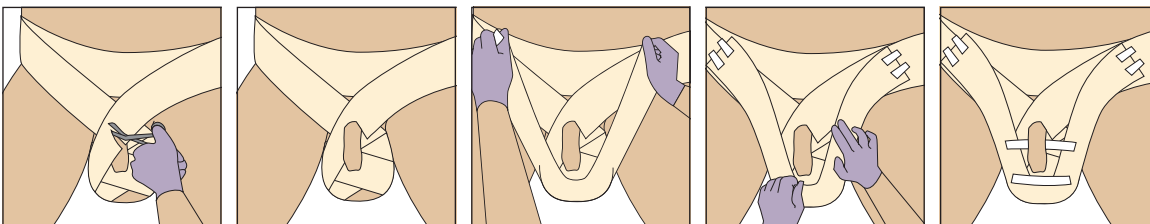


Step 7: Trim the comfort foam layer as needed to accommodate the penis.


Step 8: If needed, apply a strip of comfort foam layer to provide additional lift and support of the scrotum. Press, mould and cut at lateral sides. Secure ends with tape. Gently mould and conform the comfort layer. Tape may be used to maintain the bandage contours.

Application of compression layer for scrotum: layer 2



- ▶ Apply compression material with **50% overlap** and with enough tension to provide support and lift of oedema
- ▶ It is recommended that you hold the roll close to the body throughout the application for controlled, even compression

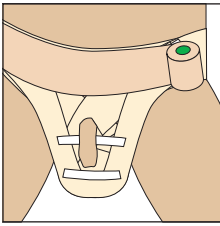



Step 1: Begin the application following the same circular winding around the hips centering over the mons pubis.

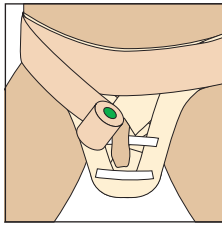


Step 2: Bring the winding distally just beneath the penis. Proceed under the scrotum and come across the top in an upward direction. Continue windings to cover all of comfort foam layer with at least two layers applied with just enough tension to conform. Press and mould to the anatomy.

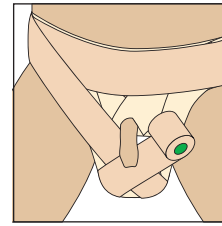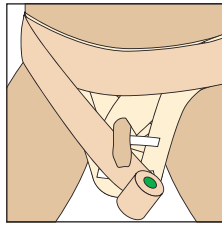




Step 3: Once scrotum is covered, bring the bandage beneath the penis laterally towards the opposite hip. At the lateral side, press, mould and cut the bandage.

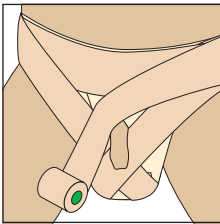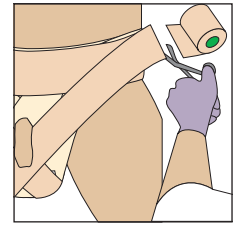


Step 4: For a complete coverage of the proximal scrotum, beginning at the lateral side, apply a strip of compression layer coming over the mons pubis ensuring that the bandage lays above the base of the penis.

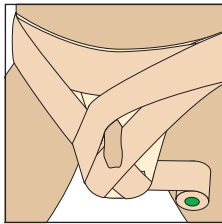


Step 5: Proceed with the bandage distally bringing the winding under the scrotum. Lift the penis and continue windings to cover the scrotum with at least two compression layers with enough tension for comfort.

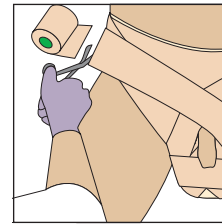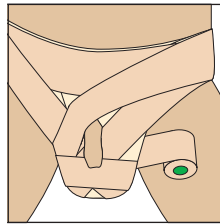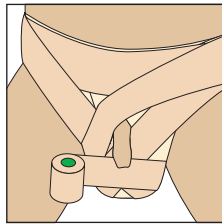



Step 6: Once scrotum is covered, bring the bandage laterally above the base of the penis, towards the opposite hip. At the lateral side, press, mould, and cut the bandage.

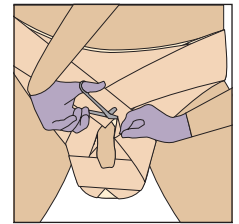


Step 7: Trim the compression layer as needed to accommodate the penis.

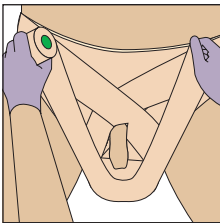

Step 8: Apply a strip of compression layer to provide additional lift and support of the scrotum. Press and cut at lateral sides. Mould the material to conform to the scrotal contours.

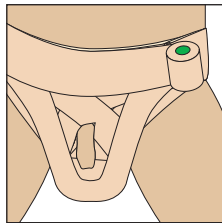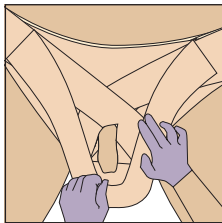

Step 9: A circular winding of compression layer can be applied to secure the cut edges.

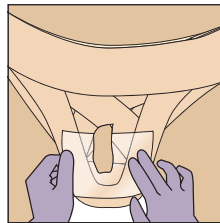


Step 10: Applying a piece of film dressing on top of the scrotum keeps the bandage clean after urination.

Materials and positioning

- ▶ Select the 2.5cm 3M™ Coban™ 2 Lite Compression Layer

Method A: application of compression layer for penis

- ▶ Apply compression material with **50% overlap** and gently guide roll to conform to anatomy
- ▶ It is recommended that you hold the roll close to the body throughout the application for controlled, even compression

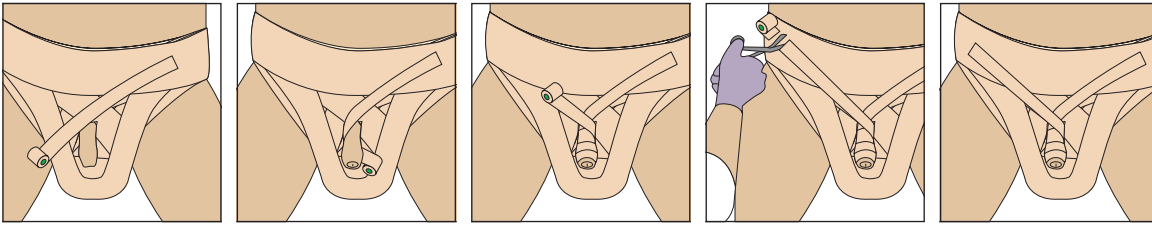

Step 1: Without tension, anchor the bandage by beginning the application with the material positioned at the lateral side of the abdominal bandage. Direct the roll across the base of the penis and under towards the distal penis.


Step 2: Proceed proximally with circular windings and **50% overlap** towards the base of penis. Anchor application by bringing the material to opposite side. Cut, press, and mould to conform.

Materials and positioning

- ▶ For extreme oedema or when additional comfort is needed, select the 7.5cm lite comfort foam layer and 2.5cm lite compression layer

Method B: application of comfort foam layer for penis: layer 1

- ▶ Apply this layer with the foam side against the skin and gently guide roll to conform to anatomy
- ▶ Cover the skin with as thin a layer as possible with no gaps

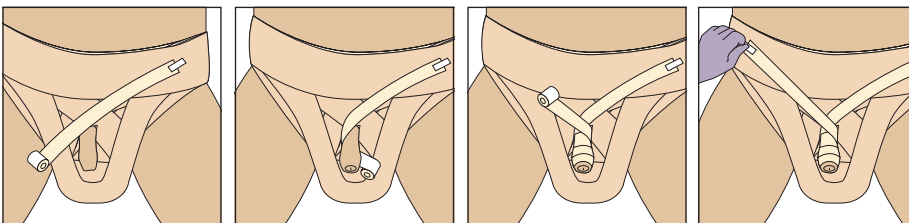


Step 1: Begin the 7.5cm comfort foam layer application with the material positioned at the lateral side of the abdominal bandage. Direct the roll across the base of the penis and under towards the distal penis.

Step 2: Proceed proximally with circular windings and minimal overlap towards the base of the penis. End the application by bringing the material to the opposite side of the abdominal bandage.

Step 3: Press, cut, and mould to conform. Secure the ends with tape.


Method B: application of compression layer for penis: layer 2

- ▶ Apply 2.5cm compression material with **50% overlap** and gently guide roll to conform to anatomy
- ▶ It is recommended that you hold the roll close to the body throughout the application for controlled, even compression

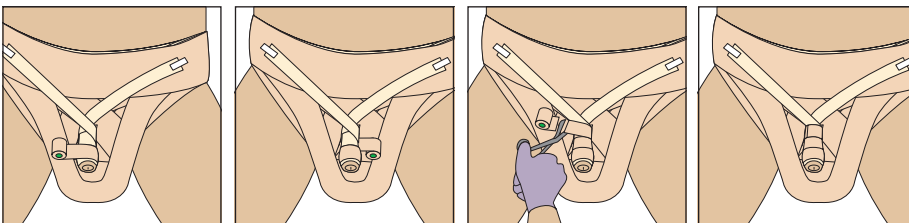

Step 1: Begin the application at the distal end of the penis and proceed proximally with circular windings at minimal stretch and **50% overlap** towards the base of the penis. End the application at the base of the penis. Press, cut, and mould.

Bandage removal

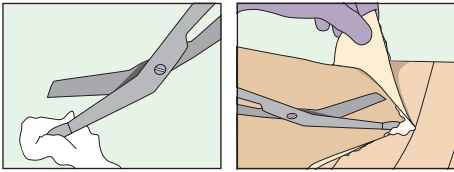

Bandage removal
Dipping the scissor tips into moisturising cream allows for comfortable and easy bandage removal.

Note

Refer to the Instructions for Use for complete information related to the use of 3M™ Coban™ 2 Lite Two-Layer Compression Systems.

For scrotum

Image	3M code	NHSSC code	PIP code	Description	Size	Compression	Box qty	Rolls per procedure
	20713/ 20723	ECA217 (BD)/ ECA221 (BD)	365-4951/ 365-4993	7.5cm Lite Individual Rolls: Comfort Layer 1/ Compression Layer 2	7.5cm x 2.7m/ 7.5cm x 3.5m	Lite	18/32 rolls	1 per application
	2794E	ECA203	355-1439	10cm Lite Kit: Comfort Layer 1 and Compression Layer 2	10cm x 2.7m and 10cm x 3.2m	Lite	2 rolls	1 per application

Alternatives

	20714/ 20724	ECA218 (BD)/ ECA222 (BD)	365-4969/ 365-5008	10cm Lite Individual Rolls: Comfort Layer 1/ Compression Layer 2	10cm x 2.7m/ 10cm x 3.5m	Lite	18/32 rolls	1 per application
--	-----------------	-----------------------------	-----------------------	--	-----------------------------	------	----------------	-------------------

For penis: methods A and B

Image	3M code	NHSSC code	PIP code	Description	Size	Compression	Box qty	Rolls per procedure
	20721	ECA220 (BD)	365-4985	2.5cm Lite Individual Rolls: Compression Layer 2	2.5cm x 3.5m	Lite	36	1 per application

Please note Blue Diamond (BD) items are not currently stocked by NHS Supply Chain. Please allow for a longer delivery lead time.

Note: Specific indications, contraindications, warnings, precautions and safety information exist for these products and therapies. Please consult a clinician and product instructions for use prior to application. This material is intended for healthcare professionals.

3M United Kingdom PLC
Charnwood Campus
10 Bakewell Road
Loughborough
LE11 5RB
+44 (0)1509 611 611

3M Ireland Limited
The Iveagh Building
Carrickmines Park
Dublin 18
+353 (0)1280 3555

© 2024 3M. All rights reserved. 3M and the other marks shown are marks and/or registered marks. Unauthorised use prohibited. Used under license in Canada. OMG960096.



View all Coban 2 Compression System application videos and guides here.

