





I.V. dressing selection guide

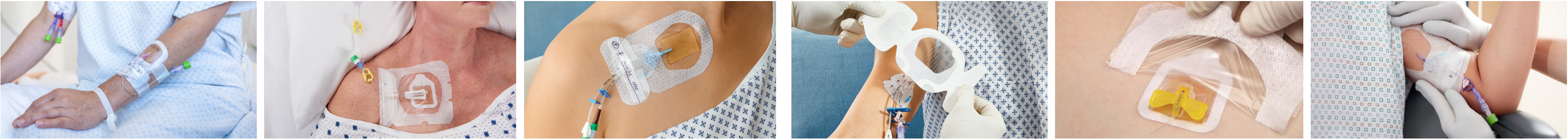

Is your dressing choice improving your patient's outcome?

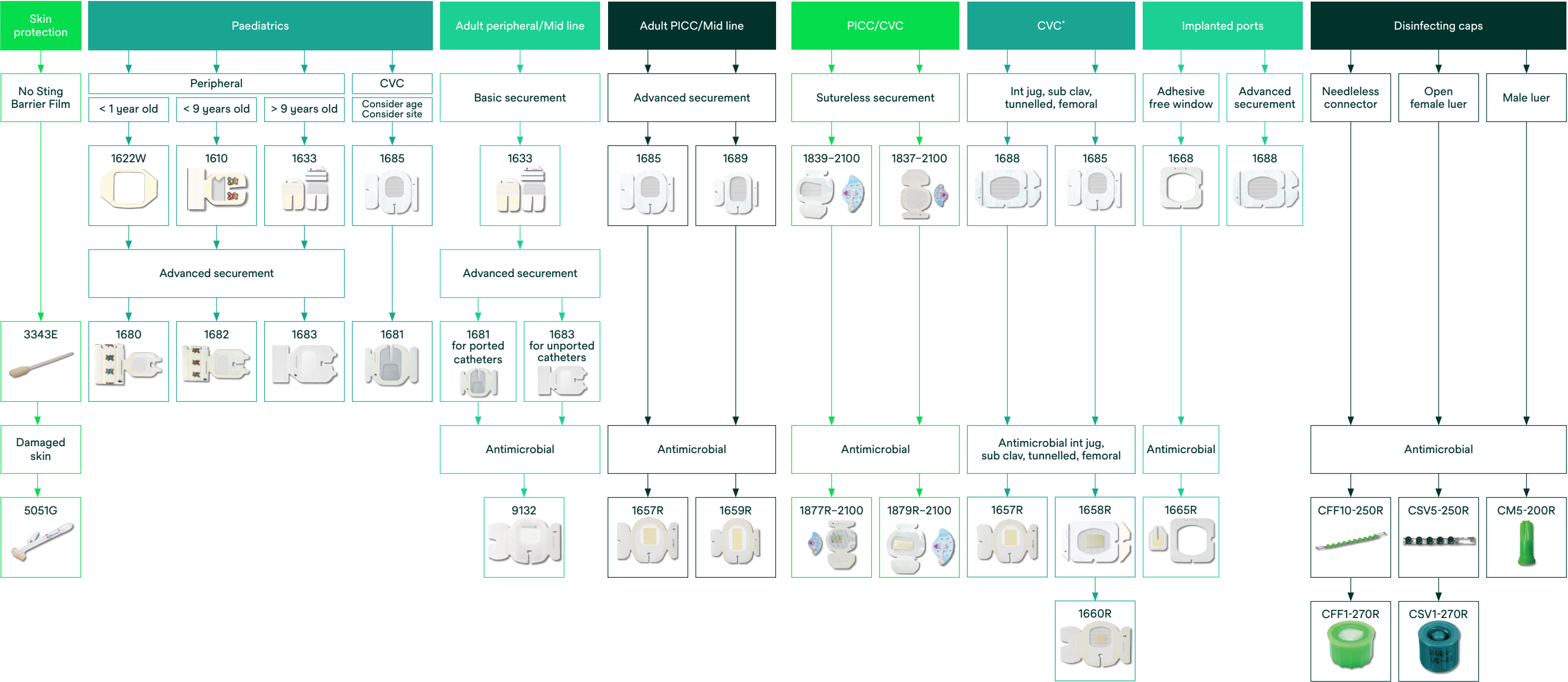

Please ensure a full and holistic assessment is made regarding selection of the appropriate vascular access device. 3M™ Tegaderm™ I.V. Dressings support up to 7 days wear time on central IV catheters and for the life of the catheter of peripheral IVs. 3M™ Curosur™ disinfectant caps provide antimicrobial protection for up to 7 days if not removed.

Please ensure that all local policies and procedures are followed.

Use clinical judgement to adapt dressing selection based on changing vascular access needs.

This dressing selection tool is a guide only.

*In addition, suggested dressings for arterial lines.

Note: specific indications, contraindications, warnings, precautions and safety information may exist for these products and therapies. Please consult a clinician and product Instructions for Use prior to application.

Considerations				
Patient Mental state, skin condition e.g. diaphoretic or fragile skin, known allergies to any components e.g. chlorhexidine.	Catheter type Number of lumens, for peripheral catheters whether ported or non-port, for huber needles wing size, tunneled vs non-tunneled, large bore catheters, etc.	Insertion site Extra security required over vulnerable areas e.g. wrist, foot, elbow, groin. Consider cannula dwell time.	Care setting Critical care, interventional radiology, haemodialysis, ER, OR, home care.	High risk patients Immunocompromised, chemotherapy, critically ill, localised signs of infection at insertion site.

Ordering information

Product code	Product description	Size	Dressings per box
Skin protection			
3343E	3M™ Cavilon™ NSBF Foam Applicator	1ml	25
5051G	3M™ Cavilon™ Advanced Skin Protectant	0.7ml	20
Antimicrobial			
1657R	3M™ Tegaderm™ CHG I.V. Securement Dressing	8.5cm x 11.5cm	25
1658R	3M™ Tegaderm™ CHG I.V. Securement Dressing	10cm x 12cm	25
1659R	3M™ Tegaderm™ CHG I.V. Securement Dressing	10cm x 15.5cm	25
9132	3M™ Tegaderm™ Antimicrobial I.V. Advanced Securement Dressing	7cm x 8.5cm	25
Advanced securement			
1681	3M™ Tegaderm™ I.V. Advanced Securement Dressing	7cm x 8cm	100
1683	3M™ Tegaderm™ I.V. Advanced Securement Dressing	6.5cm x 7cm	100
1685	3M™ Tegaderm™ I.V. Advanced Securement Dressing	8.5cm x 11.5cm	50
1688	3M™ Tegaderm™ I.V. Advanced Securement Dressing	10cm x 12cm	50
1689	3M™ Tegaderm™ I.V. Advanced Securement Dressing	10cm x 15.5cm	25
1680	3M™ Tegaderm™ I.V. Advanced Paediatric Dressing	3.8cm x 4.5cm	100
1682	3M™ Tegaderm™ I.V. Advanced Paediatric Dressing	5cm x 5.7cm	100
Standard securement			
1633	3M™ Tegaderm™ Transparent I.V. Dressing	7cm x 8.5cm	100
1610	3M™ Tegaderm™ Paediatric I.V. Dressing	5cm x 5.7cm	100
1622W	3M™ Tegaderm™ Transparent I.V. Dressing	4.4cm x 4.4cm	100
Sutureless securement for PICC and CVC catheters			
1839-2100	3M™ Tegaderm™ PICC/CVC Securement Device + I.V. Advanced	10cm x 15.5cm	20
1837-2100	3M™ Tegaderm™ PICC/CVC Securement Device + I.V. Advanced	11.5cm x 8.5cm	20
1877R-2100	3M™ Tegaderm™ PICC/CVC Securement Device + CHG	8.5cm x 11.5cm	20
1879R-2100	3M™ Tegaderm™ PICC/CVC Securement Device + CHG	10cm x 15.5cm	20
Dressings for implanted ports			
1668	3M™ Tegaderm™ I.V. Port Dressing with Adhesive-Free Window	12cm x 12cm	25
1665R	3M™ Tegaderm™ I.V. Port Dressing with CHG Gel Pad (6.2cm x 4.9cm)	12cm x 12cm	25
Disinfecting caps			
CFF1-270R	3M™ Curo™ Disinfecting Cap for needleless connectors	Singles	270*
CFF10-250R	3M™ Curo™ Disinfecting Cap for needleless connectors	Strips of 10	250*
CSV1-270R	3M™ Curo™ Stopper Disinfecting Cap for Open Female Luers	Singles	270*
CSV5-250R	3M™ Curo™ Stopper Disinfecting Cap for Open Female Luers	Strips of 5	250*
CM5-200R	3M™ Curo Tips™ Disinfecting Cap for Male Luers	Strips of 5	200*

*Number of caps



KCI Medical Limited
Charnwood Campus
10 Bakewell Road
Loughborough
LE11 5RB
01509 611 611

3M Healthcare EMEA Export Limited
2 Cumberland Place
Fenian Street
Dublin 2, Ireland
D02 H0V5
1800 333 377

www.3M.co.uk/vascularaccess

© Solventum 2025. All rights reserved. Solventum, the S logo and other trademarks are trademarks of Solventum or its affiliates. 3M, the 3M logo and other 3M owned and/or licensed trademarks are trademarks of 3M Company. OMG1953946.